

## Dräger Alcotest® 5820 Alcohol Screening Device

The Dräger Alcotest® 5820 allows the professional user to perform a breath alcohol test with speed and precision. The measuring technology of this small and user-friendly portable measuring device has already been proven in over 200,000 units in use worldwide.



USA: For Law Enforcement Purposes Only

## Benefits

---

### Breath alcohol testing made easy

The device is ready to use within seconds. This means that you can perform an (active) test on a subject at any time. It is also possible to measure alcohol in the ambient air (passive), in which case a mouthpiece isn't required.

All functions required for a measurement are activated with the convenient green OK button, while two menu buttons are used for navigation.

---

### Versatile and tough

The tried and tested electro-chemical Dräger sensor in the Alcotest 5820 is distinguished by its very fast response times and a long service life. It operates with extreme precision and reliability. The analysis is reliable even at temperatures of -5 to +50 °C / 23 to 122 °F. The sensor also delivers reliable results quickly in case of a high alcohol content, for both active and passive measurements.

---

### Convenient to use

Dealing with intoxicated people requires a high degree of attention and concentration. An intuitive operation of the device is an important aspect for ease of use and smooth breath testing procedures.

All measurement functions can be operated with just one button to make it easier for you to perform the test. The large back-lit display is easy to understand due to its full-text messages which guide you securely through the alcohol test. An LED and audible signal complement the display and indicate the end of a measurement process. Two menu buttons are used to navigate through the menu, enabling you to perform functions such as reviewing the last test results.

---

### Simply hygienic: the Slide'n'click mouthpiece

The sophisticated product design meets the requirements for performing the breath alcohol test quickly, easily and hygienically: The shape of the Slide'n'click mouthpiece allows you to intuitively fit it correctly, even in the dark. The Alcotest 5820 is ready to use again immediately after changing the mouthpiece. Furthermore, attempts at obstruction are consistently rejected by the device: The air outlet cannot be closed, foiling any attempts to manipulate the device when giving a breath sample.

A spacer on the mouthpiece keeps the lips of the person being tested from coming into contact with the device's housing. The spacer can also be used as a mouthpiece ejector if necessary. Mouthpieces with return valve are also available upon request.

## Accessories



ST-178-2004

### Mouthpieces (slide'n'click)

Available in packs of 100, 250, and 1000 mouthpieces with or without non-return valve.



D-45460-2015

### Protective cover

The cover offers additional protection. Meaning, it can be operated while in the cover, since the display is visible at all times.

## Related Products



D-74696-2013

### Dräger Alcotest® 6820

Keeping the roads safe and free of impaired drivers is an unpredictable and high stress job. Dräger designed the Alcotest® 6820 handheld breathalyzer specifically for the needs of law enforcement officers. It's easy to use and provides fast, reliable results in the toughest environments you face when every second matters.

## Related Products



D-43786-2015

### **Dräger Alcotest® 9510**

The Dräger Alcotest® 9510 is an advanced breath alcohol measuring instrument for evidential applications. It is designed to comply with national and international requirements and regulations (OIML R 126). With dual sensor technology, intuitive color touch screen interface, and modern design the Alcotest® 9510 sets superior standards.



D-42702-2015

### **Dräger DrugCheck® 3000**

Use the Dräger DrugCheck® 3000 to find out within minutes if a person recently consumed certain drugs. The compact and quick oral fluid based drug test yields reliable results affordably and easily. The device does not require electricity and can be used anywhere.



D-54720-2012

### **Dräger DrugTest® 5000**

No pipetting, no drips, no timing: Collecting an oral fluid sample is quick and easy with the Dräger DrugTest® 5000. The collected sample can then be analysed immediately for accurate results on the spot.

## Technical Data

Measuring principle	Electrochemical DrägerSensor in 1/4" technology; alcohol specific
Measuring range	0 to 2.5 mg/L; if measurement range limit is exceeded, a message is displayed
Sampling	Standard: automatic sampling when minimum volume resp. defined blow time is reached; Passive sampling without mouthpiece or manual initiation of sampling possible
Ready for use	Approx. 2 s after switching on
Display of measurement result	After approx. 3 s (at 0 mg/L); after approx. 10 s (at 0.5 mg/L, room temperature)
Operating temperature	-5 to +50 °C / 23 to 122 °F
Relative humidity	10 to 100 % relative humidity (non-condensing and in operating state)
Ambient pressure	600 to 1,300 hPa / 17.7 to 38.4 inch Hg
Display	Graphic backlit LCD display; 32 x 22 mm / 1.3" x 0.9" (128 x 64 pixels)
LED	2-colour, to support display of results and warning messages
Audible signal	Different signal tones to support display messages and warnings
Datalogger	Storage of last 100 tests with test numbers
Power supply	1 x CR123A-Battery, charge level indicator in display, with one battery approx. 1,500 breath tests can be done.
Mouthpiece adaption	Improved Slide'n'click mouthpiece attachment; can be fitted for right or left orientation
Mouthpiece	Hygienically, individually packaged, with tamper-proof, tamper-proof air outlet, mouthpiece ejector and spacer between mouth and instrument housing
Operating concept	Measurement functions can be performed using just one button; menu navigation is via two menu button
Calibration	Wet gas or dry gas calibration
Housing	Impact resistant ABS/PC
Dimensions (W x H x D), weight	Approx. 50/60 x 141 x 31 mm / 2"/2.4" x 5.6" x 1.2"; approx. 150 g / 0.33 lbs, incl. battery
Instrument configuration	Direct menu-guided configuration of instrument settings (PIN required) No additional PC software needed
Vibration and shock	EN 60068-2-27, EN 60068-2-6; EN 60068-2-64
CE marking	2014/30/EU (electromagnetic compatibility)
Norm	EN 15964, NHTSA, FDA Conformity, depending on configuration
Internal clock	Warning or deactivation after end of service interval
Protection type	IP54

## Ordering Information

Dräger Alcotest® 5820 – Configurator	83 24 921
Dräger Alcotest® 5820 – fixed configuration (device, 3 mouthpieces, 1 battery, hand loop, plastic transportation case.	83 25 200
Dräger Alcotest® 5820 – USA	83 24 940
Dräger Alcotest® 5820 – Japan	83 25 230
Mouthpiece – Package with 100 pieces	68 10 690
Mouthpiece – Package with 250 pieces	68 10 825
Mouthpiece – Package with 1,000 pieces	68 10 830

## Ordering Information

Mouthpiece with non return valve – Package with 100 pieces	68 11 055
Mouthpiece with non return valve – Package with 250 pieces	68 11 060
Mouthpiece with non return valve – Package with 1,000 pieces	68 11 065
Protection bag, black	83 24 999
Battery	45 43 808

## Notes

## Notes

Not all products, features, or services are for sale in all countries.  
Mentioned Trademarks are only registered in certain countries and not necessarily in the country in which this material is released. Go to [www.draeger.com/trademarks](http://www.draeger.com/trademarks) to find the current status.

### CORPORATE HEADQUARTERS

Drägerwerk AG & Co. KGaA  
Moislinger Allee 53–55  
23558 Lübeck, Germany  
[www.draeger.com](http://www.draeger.com)

### AUSTRALIA

#### MELBOURNE (HEAD OFFICE)

Draeger Safety Pacific Pty. Ltd.  
8 Acacia Place  
Notting Hill VIC 3168  
Tel (Local Toll-free) 1800 372 437 Tel  
+61 3 9265 5000  
Fax (Local Toll-free) 1800 647 484  
Fax +61 3 9265 5095  
[customer.service@draeger.com](mailto:customer.service@draeger.com)

### SALES AND SERVICE CENTRES

#### ADELAIDE

258 Grange Road  
Flinders Park SA 5025

### BRISBANE

Unit 7 Rocklea Place  
1909 Ipswich Rd  
Rocklea QLD 4106

### DARWIN

Unit 2/101 Koonawarra Rd  
Winnellie NT 0820

### GLADSTONE

Unit 5, 39 Toolooa St  
Gladstone QLD 4680

### KALGOORLIE

184 Hay Street  
Kalgoorlie WA 6430

### KARRATHA

Unit 1, 16 Hedland Place  
Karratha WA 6714

### PERTH

72A Irvine Drive  
Malaga WA 6090

### SYDNEY

24/7 Salisbury Rd  
Castle Hill NSW 2155

### NEW ZEALAND

#### AUCKLAND

Unit 4, 24 Bishop Dunn Place  
East Tamaki, Auckland  
Tel 0800 372 437  
Fax 0800 733 133

#### WELLINGTON

Unit 3, Parkhead Court  
33 Pito-One Road Korokoro,  
Lower Hutt 5012

Locate your Regional Sales  
Representative at:  
[www.draeger.com/contact](http://www.draeger.com/contact)

